


Poor Bond of Impression Material to the Tray

Visual Appearance: Impression pulling away from the sides/bottom of tray.

Result: Crown(s) may be tight and not seat fully, or require excessive internal adjustment.

CAUSE	SOLUTION
No tray adhesive used.	Use tray adhesive.
Incompatible tray adhesive used.	Use appropriate tray adhesive. VPS adhesive for VPS. Polyether adhesive for polyether materials.
Inadequate drying time for tray adhesive.	Follow manufacturer's instructions for application, and drying time.
Thin plastic trays allow deflection, which can cause rebound upon removal.	Use a tray that fits better, and is stiffer and more rigid.

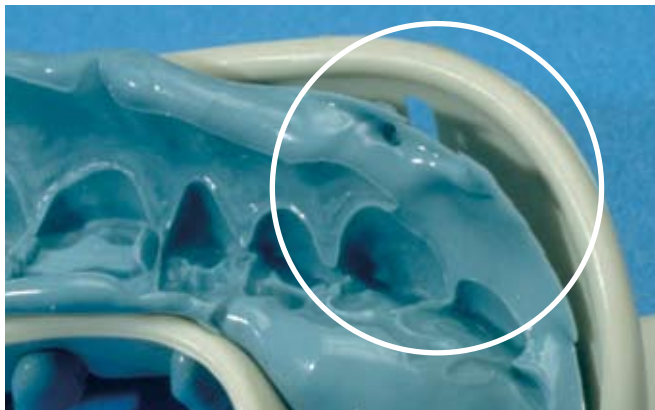

Separation of Material from Tray

3M ESPE

Making Better Impressions

Expertise™

Trouble

Shooting

Guide